

LAB ALERT – NEW TEST**NOTIFICATION DATE: 10/18/2013****EFFECTIVE DATE: 10/24/2013****CONTROLLED DRUG MONITOR****TEST CODE:** CDM02**INTERFACE CODE:** Manual orders only**METHODOLOGY:** LC/MS/MS, Spectrophotometry and Refractometry**CLINICAL UTILITY:** Please note that this test Does Not include Urine Drug Screen. If Urine Drug screen is also required then please order CDM.

This is a quantitative LC/MS/MS assay that detects, confirms and quantitates the controlled drugs and/or their metabolites in the panel by High Performance Liquid Chromatograph Tandem Mass Spectrometry. This test includes confirmation and quantitation of all of the following controlled substances and/or their metabolites: Alprazolam, Alpha-OH-alprazolam, 7-Aminoclonazepam, Nordiazepam, Diazepam, Lorazepam, Oxazepam, Temazepam, Fentanyl, Norfentanyl, Methadone, Methadone Metabolite, Tramadol, Buprenorphine, Norbuprenorphine, Morphine, 6-Acetylmorphine, Codeine, Hydrocodone, Hydromorphone, Oxycodone, Oxymorphone, and Dihydrocodeine. This test also includes Adulteration Detection Panel, i.e. measurement of urine specific gravity and urine creatinine.

PERFORMED: Tuesdays**COLLECTION:** Collect Random Urine without preservatives and keep Refrigerated.**SPECIMEN PREPARATION:** Submit 15 mL Random Urine (Minimum 5 mL) in a screw capped container.**STABILITY (FROM COLLECTION TO INITIATION OF TESTING)** Refrigerated: 7 days
Frozen: 1 month

TRANSPORT: Refrigerated

REFERENCE RANGE: Quantitative report provided if the controlled drug(s) and/or their metabolite(s) detected.

RESULTS REPORTED: 1 – 8 days

CPT CODE(S): 83840x2, 83925x4, 82646, 82649, 80154x2, 80299x3, 80102xvariable; Adulteration Detection: 82750, 81002

PERFORMING LAB: med fusion

ORDER/CONTACT INFORMATION: ***ClearPoint Client Services: 972-966-7700 | Fax: 972-966-7799***